

Детектор утечки крови BLD03/50I

Технические характеристики

По вопросам продаж и поддержки обращайтесь:

Алматы (727)345-47-04
Ангарск (3955)60-70-56
Архангельск (8182)63-90-72
Астрахань (8512)99-46-04
Барнаул (3852)73-04-60
Белгород (4722)40-23-64
Благовещенск (4162)22-76-07
Брянск (4832)59-03-52
Владивосток (423)249-28-31
Владикавказ (8672)28-90-48
Владимир (4922)49-43-18
Волгоград (844)278-03-48
Вологда (8172)26-41-59
Воронеж (473)204-51-73
Екатеринбург (343)384-55-89

Иваново (4932)77-34-06
Ижевск (3412)26-03-58
Иркутск (395)279-98-46
Казань (843)206-01-48
Калининград (4012)72-03-81
Калуга (4842)92-23-67
Кемерово (3842)65-04-62
Киров (8332)68-02-04
Коломна (4966)23-41-49
Кострома (4942)77-07-48
Краснодар (861)203-40-90
Красноярск (391)204-63-61
Курск (4712)77-13-04
Курган (3522)50-90-47
Липецк (4742)52-20-81

Магнитогорск (3519)55-03-13
Москва (495)268-04-70
Мурманск (8152)59-64-93
Набережные Челны (8552)20-53-41
Нижний Новгород (831)429-08-12
Новокузнецк (3843)20-46-81
Ноябрьск (3496)41-32-12
Новосибирск (383)227-86-73
Омск (3812)21-46-40
Орел (4862)44-53-42
Оренбург (3532)37-68-04
Пенза (8412)22-31-16
Петрозаводск (8142)55-98-37
Псков (8112)59-10-37
Пермь (342)205-81-47

Ростов-на-Дону (863)308-18-15
Рязань (4912)46-61-64
Самара (846)206-03-16
Санкт-Петербург (812)309-46-40
Саратов (845)249-38-78
Севастополь (8692)22-31-93
Саранск (8342)22-96-24
Симферополь (3652)67-13-56
Смоленск (4812)29-41-54
Сочи (862)225-72-31
Ставрополь (8652)20-65-13
Сургут (3462)77-98-35
Сыктывкар (8212)25-95-17
Тамбов (4752)50-40-97
Тверь (4822)63-31-35

Тольятти (8482)63-91-07
Томск (3822)98-41-53
Тула (4872)33-79-87
Тюмень (3452)66-21-18
Ульяновск (8422)24-23-59
Улан-Удэ (3012)59-97-51
Уфа (347)229-48-12
Хабаровск (4212)92-98-04
Чебоксары (8352)28-53-07
Челябинск (351)202-03-61
Череповец (8202)49-02-64
Чита (3022)38-34-83
Якутск (4112)23-90-97
Ярославль (4852)69-52-93

Россия +7(495)268-04-70

Казахстан +(727)345-47-04

Беларусь +(375)257-127-884

Узбекистан +998(71)205-18-59

Киргизия +996(312)96-26-47

эл.почта: sqe@nt-rt.ru || сайт: <https://sonotec.nt-rt.ru/>

General

Description

The Blood Leak Detector BLD03/50I is a non-invasive, optical sensor designed to detect smallest amounts of blood in a clear fluid through an optical transparent and flexible tubing.

The BLD03/50I consists of a sensor head with connection line and an external electronic board (PCB) with integrated firmware.

The sensor is developed as a built-in component integrated into a medical device.

Scope of delivery

	Article number
Set BLD03/50I, class A Including:	700 01 0408
Sensor head BLD03/50	200 05 0008
PCB BLD03I, programmed, class A	200 01 0338

Sensor Head BLD03/50




Technical data	
Measuring method	Optical transmission measurement, LED
Specification	Sensor head BLD03/50 - channel width 5.0 mm
Order number	200 05 0008
Materials	Housing: ABS and glass Potting: PUR (transparent)
Measuring channel	Width: 5.0 mm; Height: 5.8 mm
Mounting	Two mounting holes (Ø = 3.25 mm)
	ⓘ The sensor must be installed on a plane surface.
Operating temperature	+5 °C ... +50 °C
Storage temperature	-20 °C ... +80 °C
Working humidity	20 % ... 75 % relative humidity (not condensing)
Storage humidity	10 % ... 90 % relative humidity (not condensing)
Protection	IP67 (Sensor head without connector)
Requirements for tube	Tube is inserted into sensor without any coupling fluid.
	Tube must always be inserted completely into sensor channel.
ⓘ Please provide us with a sample of the tube (approx. 30 cm), so that we can test the usability of your tube.	
⚠ WARNING Make sure that the tube remains in position and will not slide out of the sensor channel while operation.	
Outer diameter	5.5 ... 7.0 mm
Wall thickness	≤ 1.25 mm
Material	Plastics, e.g. PVC, PE, silicone, PUR, other materials on request



Technical data

Special features	Tube must be optically transparent within the spectral range of about 415 nm
Elasticity	Tube must be able to adjust flexibly
Liquid requirements	Optically transparent liquids
Cable	Sensocord®-M-UL, 5 × AWG 28, unshielded, black, Length: 500 mm ± 20 mm
Connector	Molex PicoBlade 51021-0500, Single Row, 5 circuits, 1.25 mm
Directives/Standards	The sensors were developed to be tested with respect to the following standards: • Safety requirements: IEC 60601-2-16:2018 • EMC: IEC 60601-1-2:2014 (4th edition) + AMD1:2020
Scope of delivery	Sensor Head BLD03/50 (connection cable included) Technical data sheet
Labelling	Each sensor head is labelled with its part number and unique serial number

Technical drawing

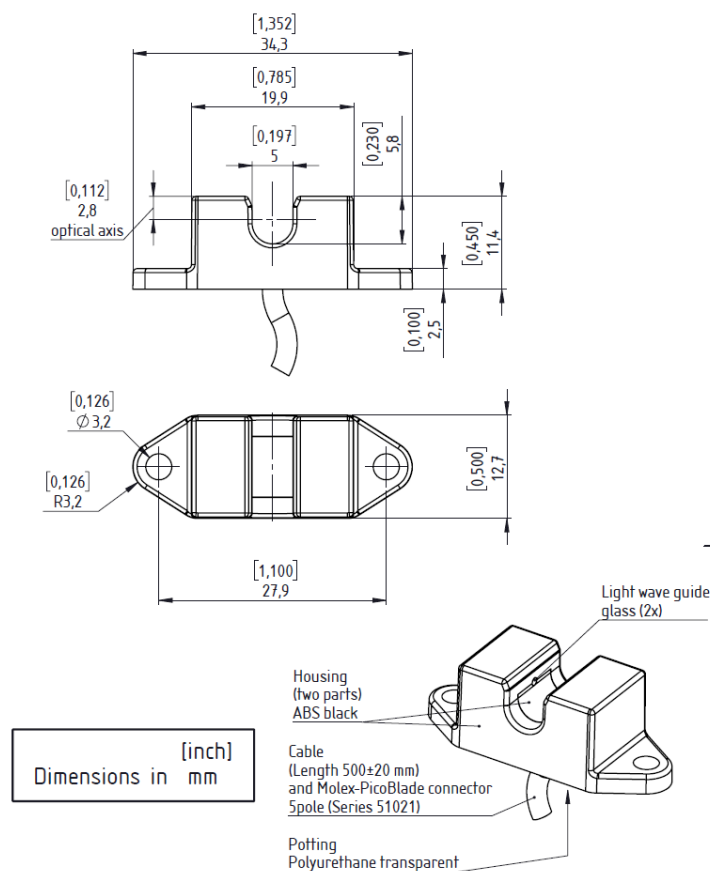

Printed Circuit Board BLD03I

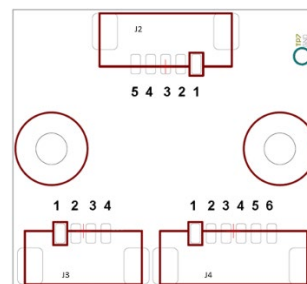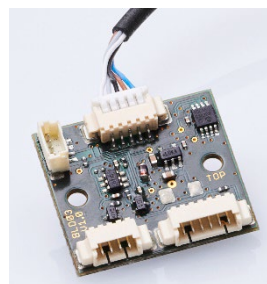

Technical data

Measuring method	Optical transmission measurement
Specification	Printed Circuit Board BLD03I V1.1, FW V04.05
Order number	200 01 0338
Blood threshold (adjustable)	Light attenuation of 8 % compared to the calibrated light intensity (default setting) ① Adjusting the blood threshold will have direct impact on blood sensitivity.
Blood sensitivity	At blood threshold of 8 % (default setting): Reliable detection of a blood leakage ≥ 0.35 ml/min at a dialysis fluid flow rate of 800 ml/min, haematocrit level of 32 %. ① Blood sensitivity is depending on tube properties, application, process characteristics and blood threshold.
Measurement accuracy	$\pm 1 \%$
Response time for blood alarm (adjustable)	Time that passes before a blood alert is actually issued in case of a blood contamination. 280 ms $\pm$ 10 ms (default setting)
Response time for plausibility test	Time that passes before a plausibility alert is actually issued in case of a non-plausible signal. 550 ms $\pm$ 20 ms
Operating voltage	+3.1 (-1.5 %) ... +5.5 (+2 %) VDC with a ripple lower than 25 mVpp ① No overvoltage protection and no reverse polarity protection implemented. Appropriate protective measures must be taken on the medical device side.
Current consumption	≤ 40 mA without additional load ① The power supply must be limited to maximum current of 200 mA by means of suitable equipment in the machine (fuse / regulator / etc.).

Technical data

Max. output current ± 50 mA

ⓘ Stresses beyond the maximum rating may cause permanent damage to the device.

Safety aspects Self-test – A self-test routine can be triggered via digital input.

Operating temperature +5 °C ... +50 °C

Storage temperature -20 °C ... +80 °C

Working humidity 20 % ... 75 % relative humidity (not condensing)

Storage humidity 10 % ... 90 % relative humidity (not condensing)

Directives/Standards The sensors were developed to be tested with respect to the following standards:

- Safety requirements: IEC 60601-2-16:2018
- EMC: IEC 60601-1-2:2014 (4th edition) + AMD1:2020
- Software is developed acc. to IEC 62304:2006 + AMD1:2015

The embedded software is classified as "A"

Scope of delivery PCB BLD03I V1.1, FW V04.05
Technical Data Sheet
Operating Manual
Serial interface Description

Labelling Each Printed Circuit Board is labelled with its product number and its unique serial number

Electrical Connection


ATTENTION

The board needs to be protected against unintended contact with other conductive parts.

J2 – Connection to Sensor Head

Connector Board connector header; 5-pin; 1.25 mm Molex: 53261-0571


Assignment	Pin	Connection
	1	GND
	2	VDD
	3	SDA of I ² C
	4	SCL of I ² C
	5	LED+

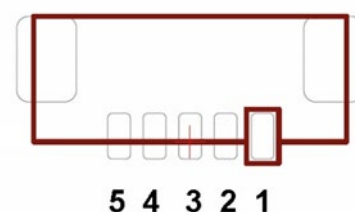

J3 – Serial Interface

Connector Board connector header; 4-pin; 1.25 mm Molex: 53261-0471


Assignment	Pin	Connection
	1	Supply voltage 3.1 ... 5.5 VDC
	2	GND
	3	Tx
	4	Rx

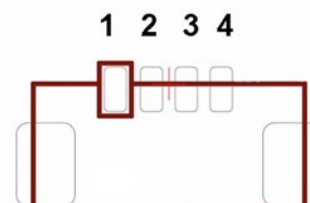

Serial interface	Parameter	Value
	Bits per second	115200
	Data bits	8
	Parity	none
	Stop bits	1
	Flow control	none

J4 – Logical Interface

Connector	Board connector header; 6-pin; 1.25 mm Molex: 53261-0671	
Assignment	Pin	Function
	1	Supply voltage 3.1 ... 5.5 VDC
	2	GND
	3	Output "Blood"
	4	Input "Self-Test" (high active)
	5	Output "Plausibility"
	6	Input "Calibration" (high active)

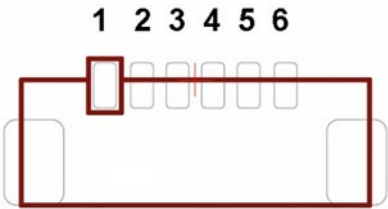

Logical specification	Output	Signal level	Condition
Signal at output	Blood	High	Blood
		Low	No blood (clear liquid)
	Plausibility	High	No plausibility error nor device error
		Low	Plausibility error or device error


① The logical output signal of “Plausibility” has the highest priority, thus signals on the logical output “Blood” are only valid if the logical output “Plausibility” is set to high.

Logical specification	Input	Signal level	Condition
Signal at input	Self-Test	High	Start self-test
		Low	-
	Calibration	High	Start calibration
		Low	-

① The logical input signal of “Calibration” has the highest priority, thus a self-test will be started only if the logical input “Self-Test” is high **and** the logical input “Calibration” is low.

Logical level	Parameter	Condition	Operating voltage V _{CC}	Min	Typ	Max	Unit
Signal at output	Output V _{OH} High Voltage	I _{OH} = -100 µA	3.1 ... 5.5 V	V _{CC} -0.1	---	---	V
		I _{OH} = -16 mA	3.1 V	2.4	---	---	V
		I _{OH} = -24 mA	3.1 V	2.3	---	---	V
		I _{OH} = -32 mA	4.5 V	3.8	---	---	V
	Output V _{OL} Low Voltage	I _{OL} = 100 µA	3.1...5.5 V	---	---	0.1	V
		I _{OL} = 16 mA	3.1 V	---	---	0.4	V
		I _{OL} = 24 mA	3.1 V	---	---	0.55	V
		I _{OL} = 32 mA	4.5 V	---	---	0.55	V
Signal at input	Input V _{IH} High Voltage		3.1 ... 5.5 V	2.0	---	---	V
	Input V _{IL} Low Voltage		3.1 ... 5.5 V	---	---	0.8	V

Technical drawing

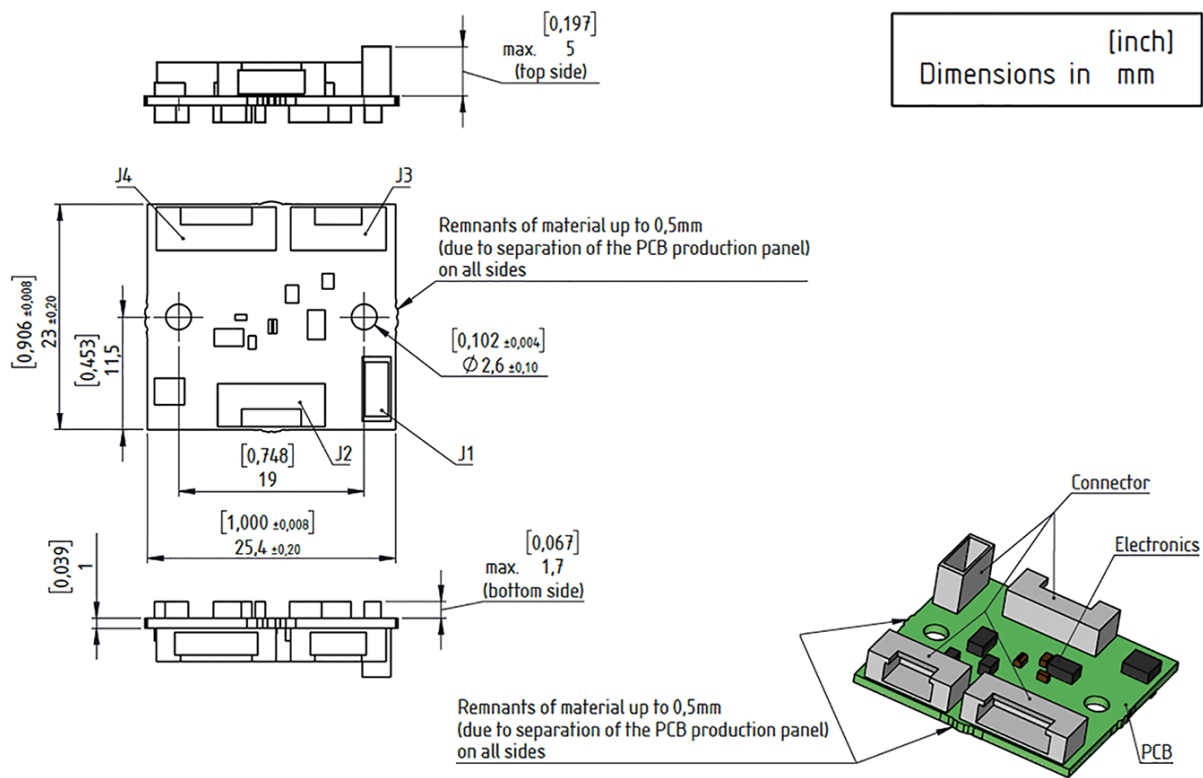



По вопросам продаж и поддержки обращайтесь:

Алматы (727)345-47-04
Ангарск (3955)60-70-56
Архангельск (8182)63-90-72
Астрахань (8512)99-46-04
Барнаул (3852)73-04-60
Белгород (4722)40-23-64
Благовещенск (4162)22-76-07
Брянск (4832)59-03-52
Владивосток (423)249-28-31
Владикавказ (8672)28-90-48
Владимир (4922)49-43-18
Волгоград (844)278-03-48
Вологда (8172)26-41-59
Воронеж (473)204-51-73
Екатеринбург (343)384-55-89

Иваново (4932)77-34-06
Ижевск (3412)26-03-58
Иркутск (395)279-98-46
Казань (843)206-01-48
Калининград (4012)72-03-81
Калуга (4842)92-23-67
Кемерово (3842)65-04-62
Киров (8332)68-02-04
Коломна (4966)23-41-49
Кострома (4942)77-07-48
Краснодар (861)203-40-90
Красноярск (391)204-63-61
Курск (4712)77-13-04
Курган (3522)50-90-47
Липецк (4742)52-20-81

Магнитогорск (3519)55-03-13
Москва (495)268-04-70
Мурманск (8152)59-64-93
Набережные Челны (8552)20-53-41
Новокузнецк (3843)20-46-81
Ноябрьск (3496)41-32-12
Новосибирск (383)227-86-73
Омск (3812)21-46-40
Орел (4862)44-53-42
Оренбург (3532)37-68-04
Пенза (8412)22-31-16
Петрозаводск (8142)55-98-37
Псков (8112)59-10-37
Пермь (342)205-81-47

Ростов-на-Дону (863)308-18-15
Рязань (4912)46-61-64
Самара (846)206-03-16
Санкт-Петербург (812)309-46-40
Саратов (845)249-38-78
Севастополь (8692)22-31-93
Саранск (8342)22-96-24
Симферополь (3652)67-13-56
Смоленск (4812)29-41-54
Сочи (862)225-72-31
Ставрополь (8652)20-65-13
Сургут (3462)77-98-35
Сыктывкар (8212)25-95-17
Тамбов (4752)50-40-97
Тверь (4822)63-31-35

Тольятти (8482)63-91-07
Томск (3822)98-41-53
Тула (4872)33-79-87
Тюмень (3452)66-21-18
Ульяновск (8422)24-23-59
Улан-Удэ (3012)59-97-51
Уфа (347)229-48-12
Хабаровск (4212)92-98-04
Чебоксары (8352)28-53-07
Челябинск (351)202-03-61
Череповец (8202)49-02-64
Чита (3022)38-34-83
Якутск (4112)23-90-97
Ярославль (4852)69-52-93

Россия +7(495)268-04-70

Казахстан +(727)345-47-04

Беларусь +(375)257-127-884

Узбекистан +998(71)205-18-59

Киргизия +996(312)96-26-47

эл.почта: sqe@nt-rt.ru || сайт: <https://sonotec.nt-rt.ru/>